

Endo it your way!

Find the VDW
endo motor
that matches
your style



VDW.GOLD® RECIPROC®

Endo motor with integrated apex locator
A market proven device you can rely on!

*Safety features
for a smooth
and successful
preparation*



- **RECIPROC REVERSE Comfort Function**:** two-stage indicator helps determine the right method of preparation
- Additional setting for root canals with difficult anatomy for rotary motion

ANA*: Reduction of torque limit during the preparation of canals with difficult anatomies

ASR*: Automatic reverse rotation when the set torque limit is reached

*in rotary mode only **in RECIPROC® mode only


High performance for rotary and reciprocating root canal preparation

- Powerful and safe preparation in torque controlled rotary and reciprocating motion
- Supports an extensive range of rotary and reciprocating file systems
- Dr's Choice: store up to 15 individual torque and speed settings
- Battery-operated motor; maintains constant operation even during charging

Powerful preparation and precise length determination in one device

Choose between three working modes:

1. Monitor the file progression simultaneously during preparation.
2. Prepare the root canal without length determination.
3. Determine the working length or monitor the file progression independently of preparation.

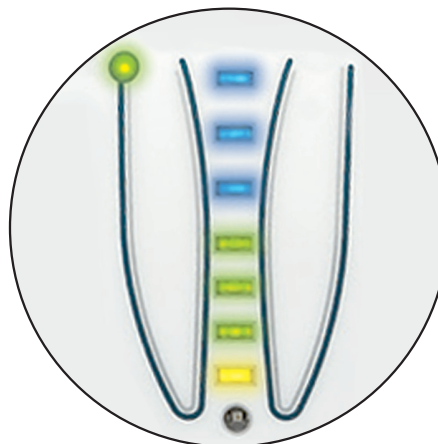

THE GOLD SUCCESS STORY

In 2007, VDW challenged the established endo standards by introducing VDW.GOLD® as the first endo motor with an integrated apex locator. In 2011, the scope was extended to the innovative preparation method RECIPROC® with the new VDW.GOLD®RECIPROC® edition. Today, VDW's GOLD motor is one of the best-selling endo motors on the market, always reflecting VDW's pure endo expertise and German heritage of engineering excellence. Continuous improvements make it one of the most reliable endo motors, coming with an exceptional 3-year guarantee. An outstanding success that VDW shares with thousands of VDW.GOLD® users all over the world.



VDW.SILVER® RECIPROC®

Stand-alone endo motor

Essential performance for effective and safe root canal preparation

- Choose from a wide range of reciprocating and rotary file systems
- Dr's Choice: store up to 15 individual torque and speed settings
- ASR*: automatic reverse rotation when the set torque limit is reached
- RECIPROC REVERSE Comfort Function**: two-stage indicator identifies the right method of preparation
- Battery-operated motor; maintains constant operation even during charging

*in rotary mode only **in RECIPROC® mode only



VDW.SILVER® RECIPROC® &
RAYPEX® 6

RAYPEX® 6

Stand-alone apex locator

Length determination with user-friendly interface

- Clear acoustic and visual signals on a bright color touch screen
- Various safety functions to enhance user and patient confidence
- Demo mode for easy and comprehensible patient education
- Intuitive handling, handy pocket-sized design, foldable
- Long-lasting rechargeable battery

A powerful duo for
professional
endo treatments



RECIPROC® direct

The first reciprocating contra-angle
It's not a big deal to start with RECIPROC®

- A quick and inexpensive way to the most time-saving preparation technique
- Simply click it on your electric or air-driven motor*
- Drives RECIPROC® instruments safely and efficiently
- Ideal accessory to your dental unit allowing you to switch easily from manual to mechanical preparation

* ISO-E coupling according to ISO 3964 is necessary



Be pragmatic:
Simply make
your dental unit
an endo unit!

My choice?
Plug'n'play!



Find the VDW endo motor that matches your style!

		VDW.GOLD® RECIPROC®	VDW.SILVER® RECIPROC®	RECIPROC® direct
File library with specific pre-settings of reciprocating and rotary NiTi systems				
Reciprocating systems	RECIPROC®, RECIPROC® <i>blue</i> , R-PILOT™	✓	✓	✓
	WaveOne®, WaveOne® Gold, WaveOne® Gold Glider	✓	✓	✗
Continuous rotary NiTi systems	Mtwo®, Mtwo® Retreatment, FlexMaster®, ProTaper®, ProTaper Next®, ProTaper® Universal, ProTaper® Retreatment, ProGlider®, PathFile®	✓	✓	✗
	K3™, Gates	✓	✓	✗
	ProFile®, GT® Series X, Hero®, RaCe, FlexMaster® Retreatment, Lentulo	✓	✗	✗
Individually programmable torque and speed settings		✓	✓	–
Individually programmable file sequences		✗	✗	–
Special safety features (in rotary mode)	ASR: Automatic reverse rotation when the set torque limit is reached	✓	✓	✗
	ANA Programme: Reduction of torque limit enhances safety during the preparation of canals with difficult anatomies	✓	✗	✗
RECIPROC REVERSE Comfort Function Two-stage indicator assists the right method of preparation (in RECIPROC® mode)		✓	✓	✗
Software update possible		✓	✓	–
Constant operation even during charging		✓	✓	–
Menu Display can be individually customized: File systems that are not in use can be deactivated.		✓	✗	–
Apex locator • Choice between simultaneous length control and separate length determination • Automatic apex stop during preparation with reciprocating and rotary systems		✓	✗	✗
6:1 contra-angle		✓	✓	–



For an endo treatment in the original reciprocating motion, VDW recommends using RECIPROC®, RECIPROC® *blue* and R-PILOT™ files in combination with the VDW motors VDW.GOLD®RECIPROC®, VDW.SILVER®RECIPROC® and RECIPROC® direct.

WaveOne®, ProTaper®, ProTaper Next®, ProGlider®, PathFile® and GT® are registered trademarks owned by Dentsply Sirona. K3™ is a trademark owned by KAVO KERR GROUP. Hero® is a registered trademark owned by Micro Mega.

Developed to
work together

VDW.GOLD®RECIPROC®

VDW.GOLD®RECIPROC® _____ V04 1173 000 000

- Incl. 6:1 contra-angle

VDW.GOLD®RECIPROC® + RECIPROC® System Kit _____ V04 1173 025 611

- 12 RECIPROC® instruments (25 mm), sterile, 6 × R25, 3 × R40, 3 × R50
- 144 RECIPROC® Paper Points, assorted, sterile, 96 × R25, 24 × R40, 24 × R50
- 60 RECIPROC® Gutta-Percha cones, assorted, 40 × R25, 10 × R40, 10 × R50
- User card, brochure, directions for use

VDW.GOLD®RECIPROC® *blue* 24 _____ V04 1173 025 624

- VDW.GOLD®RECIPROC®
- RECIPROC® *blue* System Kit 24:
 - 24 RECIPROC® *blue* instruments (25 mm), sterile, 12 × R25, 6 × R40, 6 × R50
 - 180 RECIPROC® *blue* Paper Points, assorted, sterile, 60 × R25, 60 × R40, 60 × R50
 - 60 RECIPROC® *blue* Gutta-Percha cones, assorted, 36 × R25, 12 × R40, 12 × R50
 - 1 Endo training tooth, trepanated, user card, brochure, directions for use

VDW.SILVER®RECIPROC®

VDW.SILVER®RECIPROC® _____ V04 1163 000 000

- Incl. 6:1 contra-angle

VDW.SILVER®RECIPROC® + RECIPROC® System Kit _____ V04 1163 025 611

- 12 RECIPROC® instruments (25 mm), sterile, 6 × R25, 3 × R40, 3 × R50
- 144 RECIPROC® Paper Points, assorted, sterile, 96 × R25, 24 × R40, 24 × R50
- 60 RECIPROC® Gutta-Percha cones, assorted, 40 × R25, 10 × R40, 10 × R50
- 1 Endo training tooth, trepanated
- User card, brochure, directions for use

Accessories for VDW.GOLD®RECIPROC® and VDW.SILVER®RECIPROC®

6:1 Contra-angle _____ V04 1079 000 000

Contra-angle sleeve, 5 pieces _____ V04 1107 000 512

Lip clip, 5 pieces _____ V04 1107 000 514

File clip, 2 pieces _____ V04 1107 000 515

File clip cable _____ V04 1177 000 516

Lip clip cable with ferrite ring _____ V04 1107 000 519

Charger _____ V04 1107 000 517

Foot switch _____ V04 1107 000 510

RECIPROC® direct

RECIPROC® direct Starter Kit with RECIPROC® *blue* _____ 21 mm V04 1094 021 624*

- 1 RECIPROC® direct contra-angle
- 24 RECIPROC® *blue* instruments, sterile, 12 × R25, 6 × R40, 6 × R50
- 1 spray nozzle for manual lubrication
- Directions for use, user card

25 mm
V04 1095 025 000
V04 1094 025 624*

*only for EOC and Japanese markets

RECIPROC® direct Starter Kit (not available in Europe) _____ 21 mm V04 1093 021 000

- 1 RECIPROC® direct contra-angle
- 44 RECIPROC® instruments, sterile, 32 × R25, 8 × R40, 4 × R50
- 1 spray nozzle for manual lubrication
- Directions for use, user card

25 mm
V04 1093 025 000

RECIPROC® direct Contra-Angle Set _____ V04 1091 000 000

- 1 RECIPROC® direct contra-angle
- 1 spray nozzle for manual lubrication
- Directions for use, user card

RAYPEX®6

RAYPEX®6 _____ V04 1113 000 000

- Incl. accessories

Still available: Accessories for VDW.CONNECT Drive®

6:1 contra-angle _____ V04 1185 000 504

Handpiece sleeves, 100 pieces _____ V04 1185 000 503

Charger incl. adapters _____ V04 1185 000 501

Spray nozzle for manual lubrication _____ V04 1185 000 506

Contra-angle adapter for Assistina _____ V04 1185 000 511

(corresponds with W&H REF 06037100)

Accessories for iPad®

VDW.CONNECT iPad® Stand _____ M00CO00STA000

Barrier sleeves for iPad mini® 4, 100 pieces _____ M00CO00079BAR

Barrier sleeves for iPad® 9,7", 100 pieces _____ M00CO00097BAR

Barrier sleeves for iPad® 10,5", 100 pieces _____ M00CO00105BAR

Barrier sleeves for iPad® 12,9", 100 pieces _____ M00CO00129BAR



For further information please contact your preferred distributor or visit www.vdw-dental.com.

VDW GmbH
Bayerwaldstr. 15
81737 Munich
Germany
Tel +49 (0) 89 62734-0
Fax +49 (0) 89 62734-304
info@vdw-dental.com

