

CENTRIX
AccuDose[®]
Disposable Tubes & Plugs

**When was the
last time you saw a
void on a patient's
x-ray?**



AccuDose[®]. The disposable tubes and plugs that eliminate repetitive pad-to-mouth motion by putting the entire mixture at your fingertips.

With AccuDose, you can quickly and more cost-effectively dispense any type of composite or cement while reducing voids and porosities in any restoration. And because AccuDose tubes and plugs are disposable, you completely eliminate cleanup and cross-contamination. No matter what application or viscosity material you need to dispense there is an AccuDose tube and plug to get the job done right.

- Save time by having the entire mixture of material right at your fingertips and eliminate the constant pad-to-mouth motion
- Reduce voids and porosities in any restoration
- Available in a variety of styles for specific applications
- Tubes and plugs are easy to load
- Disposable to eliminate cleanup and cross-contamination
- PhotoBloc™ orange tint blocks light and extends the life of light-activated materials
- Can be used with all syringes currently on the market
- Trusted and clinically proven for over 40 years

C-R[®] Syringes, Tubes and Plugs were invented by Centrix over 40 years ago. Centrix continues to be the world leader in providing delivery systems for all dental materials.



centrix[®]

Making Dentistry Easier.SM

Syringe all types of composites, cements, glass ionomers, impression materials and other flowable products. Choose according to need or specific application.



FILL



LOAD



EXPRESS

How to Use:

1. Select appropriate tube for material.
2. Place material on mixing pad.
3. Backload material into tube. Do not load more than 1/2 full.
4. Insert plug fully into tube.
5. Place tube into the syringe barrel.
6. Squeeze syringe with slow, steady pressure.
7. Dispose of tube. Sterilize syringe.

ORDERING INFORMATION



ACCUDOSE[®] 19GA NEEDLETUBESTM (GREEN)

A 19ga bendable blunt needle for the most precise placement of cements, base materials, core buildup materials and gel-type materials.

REF 290032 100
REF 290035 500

ACCUDOSE 20GA NEEDLETUBES (ORANGE)

Reduce voids and porosities in any restoration with the 20ga bendable blunt needle. PhotoBloc™ orange tint blocks light and extends the life of light-activated materials.

REF 290031 100
REF 290033 500

ACCUDOSE 24GA NEEDLETUBES (AQUA BLUE)

The longer 24ga bendable needle can syringe many types of low viscosity materials, such as composites, cements, glass ionomers, impression materials and other flowable products deep into the narrowest preps.

REF 290049 100



ACCUDOSE LOW VISCOSITY

An all-purpose tip for medium viscosity, flowable materials such as composites, core materials, glass ionomers and cements. The tip can be clipped to a larger opening if needed.

REF 210021 100
REF 210023 500

ACCUDOSE HIGH VISCOSITY


A tip for high viscosity, heavy materials such as thick composites and core buildup materials. Large opening allows placement of even the heaviest-bodied materials into small areas.

REF 280021 100
REF 280023 500


ACCUDOSE 19GA RIBBONTUBES

A tip with a crimped, flat end that dispenses a thin and wide ribbon of material. Excellent for placing luting composite into veneer and direct composite bonding onto anterior teeth.

REF 290041 100

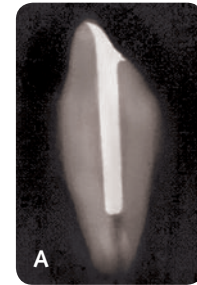

A

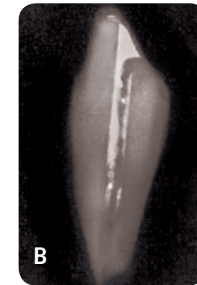

B

Fig. A - Placing materials with C-R Tubes & Plugs creating void-and porosity-free restorations.

Fig. B - Placing material traditionally with composite placement instruments, spatulas or spiral paste fillers.

Ordering is easy. Call 800.235.5862 or contact your preferred dealer.
Visit centrixdental.com for more information.

